

Supplemental Assay Testing Chart

Test	Methodology/ Platform	Sample Requirements	Testing Facility	Testing Schedule	Test Code
ALT	Kinetic enzymatic	Serum or plasma (<u>not</u> EDTA, citrate or oxalate) 1.0 mL, separate from cells <u>within 45 minutes of collection</u>	LabCorp, Dublin	Daily; TAT 1 day	001545
Babesia NAT	Roche cobas 8800	1.1 mL Whole Blood collected into Roche Whole Blood Collection Tube, up to 60 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6006515
Babesia NAT IDT	Grifols Panther	EDTA Whole Blood 3.0 mL, up to 13 days at 2-8 C. Send refrigerated.	CTS	Daily; TAT 24 hrs.	N/A
Captia Syphilis	Trinity	Serum 1.0 mL, up to 5 days 2-8 C or > 5 days frozen ≤ -20 C Plasma 1.0 mL, up to 48 hrs. 2-8 C, do not freeze plasma	Versiti DTL	Weekly; TAT 10 days	L6004380
Chagas antibody	Abbott Alinity s	Serum or plasma 1.0 mL, up to 7 days at 15 - 30 C up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6006800
Chagas antibody	Ortho <i>T. cruzi</i> / EIA	Serum or plasma 1.0 mL, up to 10 days 2-8 C	♦ Coda / Alpha: CTS→Quest ISBT: NBTC	Daily; TAT 24 hrs.	N/A
Chagas ESA	Abbott Prism	Serum or plasma 1.0 mL, 14 days at 2-8 C or up to 2 mo. ≤ -20 C	CTS	Weekly; TAT 14 days	N/A
CMV IgG	Immunoassay	Serum 1.0 mL, up to 14 days, RT or 2-8 C	LabCorp, Dublin	M-F; TAT \leq 48 hrs.	006494
CMV IgM	Immunoassay	Serum 1.0 mL, up to 14 days, RT or 2-8 C	LabCorp, Dublin	M-F; TAT \leq 48 hrs.	096727
CMV by PCR Qualitative	RT-PCR	EDTA plasma 0.5 mL, separated within 24 hrs. and frozen	LabCorp, Dublin (→Burlington, NC)	Test date varies TAT 3-4 days	138693
COVID-19 CoV-2 IgG II Antibody	Chemiluminescent microparticle immunoassay (CMIA)	4.0 mL EDTA or Clot Whole Blood, or 2 mL Serum or EDTA Plasma, up to 2 days at 15-30 C, up to 7 days at 2-8 C,	MRN Diagnostics, Franklin, MA	M-F; TAT 2-7 days	9088
EBV IgG	Immunoassay	Serum only 1.0 mL, up to 14 days 2-8 C	LabCorp, Dublin	M-F; TAT \leq 48 hrs.	096230
EBV IgM	Immunoassay	Serum only 1.0 mL, up to 14 days 2-8 C	LabCorp, Dublin	M-F; TAT \leq 48 hrs.	096735
EBV Nuclear Ag (EBNA) IgG	Chemiluminescent Immunoassay (CLIA)	Serum only 1.0 mL, up to 14 days 2-8 C	LabCorp, Dublin	M-F; TAT \leq 48 hrs.	010272
EBV by PCR Qualitative	PCR	EDTA Whole Blood 1.0 mL, up to 3 days 2-8 C	LabCorp, Dublin (→Burlington, NC)	Wed; TAT 24 hrs.	138289
Factor V Leiden	PCR	EDTA Whole Blood 3.0 mL, up to 1 week 2-8 C	LabCorp, Dublin (→RTP/Durham, NC)	Tue, Fri; TAT 5-7 days.	511154

Supplemental Assay Testing Chart

Test	Methodology /Platform	Sample Requirements	Testing Facility	Testing Schedule	Test Code
FTA ABS	IFA	Serum only 1.0 mL	Quest, VA	M-F; TAT 24 hrs.	4112
GC/ Chlamydia	NAA	Urine transferred to Aptima® Urine Specimen Collection Kit within 24 hrs.	LabCorp, Dublin (→Charleston, WV)	M-F; TAT 48 hrs.	183194
HBc antibody	Abbott Alinity s	Serum or plasma 1.0 mL, up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6000405
HBc antibody	Ortho / EIA	Serum or plasma 1.0 mL, up to 10 days 2-8 C	◆Coda / Alpha: CTS→Quest ISBT: NBTC	Daily; TAT 24 hrs.	N/A
HBc IgM (ICMA)	Immunochemi-luminometric	Serum or plasma 1.0 mL, up to 14 days 2-8 C, RT, or frozen	LabCorp, Dublin	Sun-Sat; TAT ≤ 48 hrs.	016881
HBsAB Qualitative	Immunochemi-luminometric	Serum or plasma 1.0 mL, up to 14 days 2-8 C, frozen, or RT	LabCorp, Dublin	Sun-Sat; TAT ≤ 48 hrs.	006395
HBsAg	Abbott Alinity s	Serum or plasma 1.0 mL, up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6000105
HBsAg	Ortho Bio-Rad / EIA	Serum, plasma, or cadaveric serum 1.0 mL, up to 7 days 2-8 C	◆Coda / Alpha: CTS ISBT: NBTC	Daily; TAT 24 hrs.	N/A
HBsAg Neutralization	Abbott Alinity s	Serum or plasma 1.0 mL in conical tube, up to 14 days 2-8 C	Versiti DTL	M,W,Th,F; TAT 24 hrs.	L6008600
HBsAg Neutralization	Ortho Bio-Rad / EIA	Serum or plasma 1.0 mL, up to 7 days 2-8 C, ≤ 5 freeze / thaw cycles or > 7 days frozen ≤ -20 C	◆Coda / Alpha: CTS ISBT: NBTC	Daily; TAT 24 hrs.	N/A
HBV NAT	Roche cobas 8800	Serum or plasma 1.0 mL, centrifuged within 72 hrs. up to 12 days at 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6009890
dHBV Ultrio discrim	Grifols Panther	Serum or plasma 1.0 mL, centrifuged within 72 hrs. 13 days at 2-8 C	CTS	Daily; TAT 24 hrs.	N/A
dHBV X3	Grifols Panther	Serum or plasma 3.0 mL, centrifuged within 72 hrs. 13 days at 2-8 C	CTS	Daily; TAT 24 hrs.	N/A
hCG-beta Quantitative	ECLIA Immunoassay	Serum only 1.0 mL, up to 1 week 2-8 C	LabCorp, Dublin	Sun-Sat; TAT 24 hrs.	004416
HCV antibody	Abbott Alinity s	Serum or plasma 1.0 mL, up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6000605
HCV antibody	Ortho / EIA	Serum or plasma 1.0 mL, up to 10 days 2-8 C	◆Coda / Alpha: CTS ISBT: NBTC	Daily; TAT 24 hrs.	N/A

Supplemental Assay Testing Chart

Test	Methodology /Platform	Sample Requirements	Testing Facility	Testing Schedule	Test Code
HCV Genotype	PCR	Plasma (preferred) or serum 3.0 mL, centrifuge and separate cells within 24 hrs. of collection, send frozen (preferred)	LabCorp, Dublin (→Burlington, NC)	Wed; TAT 3-5 days	550475
HCV NAT	Roche cobas 8800	Serum or plasma 1.0 mL, centrifuged within 72 hrs. up to 12 days at 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6009890
dHCV Ultrio discrim	Grifols Panther	Serum or plasma 1.0 mL, centrifuged within 72 hrs. 13 days at 2-8 C	CTS	Daily; TAT 24 hrs.	N/A
HIV-1/2/O antibody	Abbott Alinity s	Serum or plasma 1.0 mL, up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6001015
HIV-1/2/O antibody	Ortho Bio-Rad / EIA	Serum, plasma ,or cadaveric serum, 1.0 mL, up to 7 days 2-8 C	◆Coda / Alpha: CTS ISBT: NBTC	M – F; TAT 48 hrs.	N/A
HIV NAT	Roche cobas 8800	Serum or plasma 1.0 mL, centrifuged within 72 hrs. up to 12 days at 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6009890
dHIV Ultrio discrim	Grifols Panther	Serum or plasma 1.0 mL, centrifuged within 72 hrs. 13 days at 2-8 C	CTS	Daily; TAT 24 hrs.	N/A
HIV-1/2 Supplemental	Bio-Rad Geenius	Serum or plasma 0.5 mL, 7 days at 2-8 C. 48 hours at Room Temp, or > 7 days frozen ≤ -20 C. ≤ 5 freeze/thaw cycles	Versiti DTL	Weekly; TAT 10 days	N/A
HTLV-I/II antibody	Abbott Alinity s	Serum or plasma 1.0 mL, up to 14 days 2-8 C	Versiti DTL	Daily; TAT 24 hrs.	L6000815
HTLV-I/II antibody	Ortho Avioq / EIA	Serum or plasma 1.0 mL, up to 14 days 2-8 C	◆Coda / Alpha: CTS→Quest ISBT: NBTC	Daily; TAT 24 hrs.	N/A
HTLV-I/II Western Blot	MP Diagnostics	1.0 mL of serum or plasma stored at 2-8 C if test to be run within 7 days of collection, or frozen at ≤ -20 C if test is to be delayed for more than 7 days. ≤ 5 freeze thaw cycles	CTS	Weekly; TAT 14 days	N/A
HTLV by PCR Qualitative	RT-PCR (Viracor-Eurofins)	EDTA Whole Blood 2.0 mL, up to 4 days	LabCorp, Dublin (→Viracor)	M–Sat; TAT 48 hrs. (receipt at Viracor lab)	823653
NAT Ultrio x1	Grifols Panther	Serum or plasma 3.0 mL, centrifuged within 72 hrs. 13 days at 2-8 C	CTS	Daily; TAT 24 hrs.	N/A
Neisseria gonorrhoea/ Chlamydia	NAA	Urine transferred to Aptima® Urine Specimen Collection Kit within 24 hrs.	LabCorp, Dublin (→Charleston, WV)	M-F; TAT 48 hrs.	183194
Parvo IgG/IgM	Immunoassay	Serum 2.0 mL, up to 7 days 2-8 C	Quest, VA	TAT 2-3 days	8946

Supplemental Assay Testing Chart

Test	Methodology / Platform	Sample Requirements	Testing Facility	Testing Schedule	Test Code
Parvo by PCR Qualitative	PCR	EDTA Whole Blood 1.0 mL, up to 7 days	LabCorp, Dublin (→Burlington, NC)	TAT 4-5 days	138644
RPR Card/ RPR titer	Charcoal Flocculation	Serum only 1.0 mL (min. 0.5 mL), Store at room temp, refrigerated or frozen up to 14 days, 3x freeze/thaw cycles max.	LabCorp, Dublin	M-Sat; TAT 48 hours	006072/006460
SPE	Electrophoresis	Serum Protein – Serum only 2.0 mL, RT for 4 days, 2-8 C for 7 days, and frozen for 28 days.	Quest Local (→KS)	Daily; TAT 24 hrs.	0000747
Strongyloides IgG	EIA	Serum only 0.5 mL, within 16 days of collection	LabCorp, Dublin (→Burlington, NC)	TAT 1-5 days	164000
Syphilis PK-TP HA	Beckman Coulter	Serum or plasma 3.0 mL, up to 7 days at 2-8 C Serum or plasma can be used for repeat testing Serum or plasma may be stored frozen at < -20 C for up to one month. Avoid more than 5 freeze / thaw cycles.	Versiti DTL	Daily; TAT 24 hrs.	L6004350
Toxoplasmosis IgG	Immunoassay	Serum only 1.0 mL, up to 14 days refrigerated	LabCorp, Dublin	M W F; TAT 24 hrs.	006478
Toxoplasmosis IgM	Immunoassay	Serum only 1.0 mL, up to 14 days refrigerated	LabCorp, Dublin	M W F; TAT 24 hrs.	096651
Triglyceride	Colorimetric	Serum only 0.5 mL, refrigerate 2-8 C up to 7 days	Quest Local (→IL)	Daily; TAT 24 hrs.	0000896
WNV NAT	Grifols Panther	Plasma only 3.0 mL, centrifuged within 72 hrs. 8 days at 2-8 C or frozen ≤ -20 C	CTS	Daily; TAT 24 hrs.	N/A
WNV NAT	Roche cobas 8800	Plasma only 1.0 mL, centrifuged within 72 hrs. 12 days at 2-8C	Versiti DTL	Daily; TAT 24 hrs.	L6004120

Turn Around Time (TAT) note: Supplemental results are sent Monday through Friday during business hours.

◆ Codabar (coda)/ Alphanumeric (alpha) barcode examples: “P,” “JJ,” “UK,” “79,” etc. ISBT barcode examples: “W0407,” “W0363,” etc.